



Technical HL7 Conformance Profile

ENTERPRISE IMAGING OUTBOUND ORU_R01 - FORMATTED TEXT {ORC-OBR}-OBX

Copyright notice:

Copyright 2012 Agfa HealthCare

All rights reserved.

Agfa shall not be liable for errors contained herein or for incidental or consequential damages in connection with the furnishing, performance or use of this publication.

Agfa reserves the right to revise this publication and to make changes to its content at any time, without obligation to notify any person or entity of such revisions and changes.

This publication may only be used in connection with the promotion, sales, installation and use of Agfa equipment by Agfa personnel.

Publication date:

8-2015

Publication number:

V1.0 (technical)

Corporate address:

Agfa HealthCare
SIV Connectivity

Septestraat 27

B-2640 Mortsel

Belgium

+32 3 444 7588

<http://www.agfahealthcare.com/>

email : connectivity@agfa.com



Message Profile

Conformance parameters

Message Profile

- HL7 Version: 2.4
- Profile Type: HL7
- Topics: confsig-Agfa Healthcare-2.4-profile-accNE_accAL-Deferred

Encoding Method

ER7



Interaction 1

Dynamic Definition

- Accept Acknowledgement: NE
- Application Acknowledgement: AL
- Acknowledgement Mode: Deferred

Static Definition

- Event Description: ORU/ACK - Unsolicited transmission of an observation message
- Message Type: ORU
- Trigger Event: R01
- Message Structure: ORU_R01
- Topics: confsig-Agfa Healthcare-2.4-static-ORU-R01-null-ORU_R01-V1.0-DRAFT-Sender

Message structure

MSH { [**PID** [**NTE**] [**PV1** {[**PV2**]}]] { **ORC OBR** [**ZDS**] {[**DG1**]} {[**NTE**]} { Segment that repeats for every report and addenda ONLY in the last ORDER_OBSERVATION will there be an OBSERVATION segment **OBX** {[**NTE**]} } } }

MSH - Message Header

- Usage: Required
- Cardinality:1..1

Seq.	Name	Opt.	Card.	Type	Table	Contents
1	Field Separator	R	1..1	ST		
2	Encoding Characters	R	1..1	ST		
3	Sending Application	R	1..1	HD	HL70361	
3.1	namespace ID	R	1..1	IS	HL70363	
3.2	universal ID	O	0..	ST		
3.3	universal ID type	O	0..	ID	HL70301	
4	Sending Facility	R	1..1	HD	HL70362	
4.1	namespace ID	R	1..1	IS	HL70363	
4.2	universal ID	O	0..	ST		
4.3	universal ID type	O	0..	ID	HL70301	
5	Receiving Application	R	1..1	HD	HL70361	
5.1	namespace ID	R	1..1	IS	HL70363	
5.2	universal ID	O	0..	ST		
5.3	universal ID type	O	0..	ID	HL70301	
6	Receiving Facility	R	1..1	HD	HL70362	
6.1	namespace ID	R	1..1	IS	HL70363	
6.2	universal ID	O	0..	ST		
6.3	universal ID type	O	0..	ID	HL70301	
7	Date/Time Of Message	R	1..1	TS		
7.1	Date/Time	R	1..1	NM		e.g. 20040328134602+0600



Seq.	Name	Opt.	Card.	Type	Table	Contents
8	Security	O	0..1	ST		
9	Message Type	R	1..1	CM_MSG	HL70076	
9.1	message type	R	1..1	ID	HL70076	
9.2	trigger event	R	1..1	ID	HL70003	
9.3	message structure	O	0..	ID	HL70354	
10	Message Control ID	R	1..1	ST		
11	Processing ID	R	1..1	PT		
11.1	processing ID	R	1..1	ID	HL70103	
11.2	processing mode	O	0..	ID	HL70207	
12	Version ID	R	1..1	VID	HL70104	
12.1	version ID	R	1..1	ID	HL70104	
12.2	internationalization code	O	0..	CE	HL70399	
12.2.1	identifier	O	0..	ST		
12.2.2	text	O	0..	ST		
12.2.3	name of coding system	O	0..	IS	HL70396	
12.2.4	alternate identifier	O	0..	ST		
12.2.5	alternate text	O	0..	ST		
12.2.6	name of alternate coding system	O	0..	IS	HL70396	
12.3	international version ID	O	0..	CE		
12.3.1	identifier	O	0..	ST		
12.3.2	text	O	0..	ST		
12.3.3	name of coding system	O	0..	IS	HL70396	
12.3.4	alternate identifier	O	0..	ST		
12.3.5	alternate text	O	0..	ST		
12.3.6	name of alternate coding system	O	0..	IS	HL70396	
13	Sequence Number	O	0..1	NM		
14	Continuation Pointer	O	0..1	ST		
15	Accept Acknowledgment Type	O	0..1	ID	HL70155	
16	Application Acknowledgment Type	O	0..1	ID	HL70155	
17	Country Code	O	0..1	ID	HL70399	
18	Character Set	O	0..*	ID	HL70211	
19	Principal Language Of Message	O	0..1	CE		
19.2	text	O	0..	ST		
19.3	name of coding system	O	0..	IS	HL70396	
19.4	alternate identifier	O	0..	ST		
19.5	alternate text	O	0..	ST		
19.6	name of alternate coding system	O	0..	IS	HL70396	



Seq.	Name	Opt.	Card.	Type	Table	Contents
20	Alternate Character Set Handling Scheme	O	0..1	ID	HL70356	
21	Conformance Statement ID	O	0..*	ID	HL70449	

1. Field Separator

2. Encoding Characters

3. Sending Application

4. Sending Facility

5. Receiving Application

6. Receiving Facility

7. Date/Time Of Message

7.1. Date/Time

If Agility is configured to send timezone information, datetime format

YYYYMMDD[HHHMM[SS]][+ZZZZ] is used otherwise YYYYMMDD[HHHMM [SS]] is used

8. Security

9. Message Type

10. Message Control ID

11. Processing ID

12. Version ID

13. Sequence Number

14. Continuation Pointer

15. Accept Acknowledgment Type

16. Application Acknowledgment Type

17. Country Code

18. Character Set

19. Principal Language Of Message

20. Alternate Character Set Handling Scheme

21. Conformance Statement ID



Segment group: PATIENT RESULT

- Usage: Required
- Cardinality:1..*

Segment group: PATIENT

- Usage: Optional
- Cardinality:0..1

PID - Patient identification

- Usage: Required
- Cardinality:1..1

Seq.	Name	Opt.	Card.	Type	Table	Contents
1	Set ID - PID	O	0..1	SI		
3	Patient Identifier List	R	1..*	CX		
3.1	ID	R	1..1	ST		
3.4	assigning authority	R	1..1	HD		
3.4.1	namespace ID	R	1..1	IS	HL70363	
3.4.2	universal ID	O	0..	ST		
3.4.3	universal ID type	O	0..	ID	HL70301	
3.5	identifier type code (ID)	R	1..1	ID	HL70203	
5	Patient Name	R	1..*	XP		
5.1	family name	R	1..1	FN		
5.1.1	surname	R	1..1	ST		
5.2	given name	R	1..1	ST		
5.3	second and further given names or initials thereof	O	0..	ST		
5.4	suffix (e.g., JR or III)	O	0..	ST		
5.5	prefix (e.g., DR)	O	0..	ST		
5.7	name type code	O	0..	ID	HL70200	
5.8	Name Representation code	O	0..	ID	HL70465	
5.11	name assembly order	O	0..	ID	HL70444	
7	Date/Time Of Birth	O	0..1	TS		
7.1	Date/Time	R	1..1	NM		e.g. 20040328
8	Administrative Sex	R	1..1	IS	HL70001	
11	Patient Address	O	0..*	XAD		
11.1	street address (SAD)	O	0..	SAD		
11.1.1	street or mailing address	O	0..	ST		
11.1.2	street name	O	0..	ST		
11.1.3	dwelling number	O	0..	ST		
11.2	other designation	O	0..	ST		



Seq.	Name	Opt.	Card.	Type	Table	Contents
11.3	city	O	0..	ST		
11.4	state or province	O	0..	ST		
11.5	zip or postal code	O	0..	ST		
11.6	country	O	0..	ID		
11.7	address type	O	0..	ID	HL70190	
11.8	other geographic designation	O	0..	ST		
11.9	county/parish code	O	0..	IS		
13	Phone Number - Home	O	0..*	XTN		
13.1	[(999)] 999-9999 [X99999][C any text]	C	0..	TN		
13.2	telecommunication use code	R	1..1	ID	HL70201	
13.3	telecommunication equipment type (ID)	R	1..1	ID	HL70202	
13.4	Email address	O	0..	ST		
14	Phone Number - Business	O	0..*	XTN		
14.1	[(999)] 999-9999 [X99999][C any text]	C	0..	TN		
14.2	telecommunication use code	R	1..1	ID	HL70201	
14.3	telecommunication equipment type (ID)	R	1..1	ID	HL70202	
14.4	Email address	O	0..	ST		
18	Patient Account Number	O	0..1	CX		
18.1	ID	R	1..1	ST		
18.4	assigning authority	O	0..	HD		
18.4.1	namespace ID	R	1..1	IS	HL70363	
18.5	identifier type code (ID)	R	1..1	ID	HL70203	
19	SSN Number - Patient	O	0..1	ST		
23	Birth Place	O	0..1	ST		

1. Set ID - PID

3. Patient Identifier List

5. Patient Name

5.8. Name Representation code

A or empty = Alphabetic, I = Ideographic and P = Phonetic

7. Date/Time Of Birth

7.1. Date/Time

YYYYMMDD

8. Administrative Sex

M = Male F = Female U = Unknown



11. Patient Address

11.1.1. street or mailing address

This is used when SAD-2 (Street Name) is empty

11.3. city

If this is not filled in a city with code UNKNOWN will be used

11.4. state or province

Is always the highest region level, but there can be a lower subdividing region level for a state or province.

11.7. address type

At the moment O (office), H (home address) are supported.

L (Legal Address) defaults to office and the rest defaults to H (home address)

13. Phone Number - Home

13.1. [(999)] 999-9999 [X99999][C any text]

In case of telephone number/fax number/cellular number, the phone number is put in this field

13.2. telecommunication use code

Required. If this is empty the communication cannot be registered.

At the moment the ones supported are: Primary Residence Number (PRN), Work Number (WPN) and Network (email) Address (NET)

13.3. telecommunication equipment type (ID)

Required. If this is empty the communication cannot be registered.

At the moment the ones supported are: Telephone (PH), Fax (FX), Cellular Phone (CP), Internet Address (Internet)

13.4. Email address

In case of Network Email Address, this needs to be filled in.

14. Phone Number - Business

14.1. [(999)] 999-9999 [X99999][C any text]

In case of telephone number/fax number/cellular number, the phone number is put in this field

14.3. telecommunication equipment type (ID)

Required. If this is empty the communication cannot be registered.

At the moment the ones supported are: Telephone (PH), Fax (FX), Cellular Phone (CP), Internet Address (Internet)

14.4. Email address

In case of Network Email Address, this needs to be filled in.



18. Patient Account Number

19. SSN Number - Patient

23. Birth Place

NTE - Notes and Comments

- Usage: Optional

- Cardinality:0..1

Seq.	Name	Opt.	Card.	Type	Table	Contents
1	Set ID - NTE	O	0..1	SI		
2	Source of Comment	O	0..1	ID	HL70105	
3	Comment	O	0..*	FT		
4	Comment Type	O	0..1	CE	HL70364	
4.3	name of coding system	O	0..	IS	HL70396	
4.4	alternate identifier	O	0..	ST		
4.5	alternate text	O	0..	ST		
4.6	name of alternate coding system	O	0..	IS	HL70396	

1. Set ID - NTE

2. Source of Comment

3. Comment

4. Comment Type

Segment group: VISIT

- Usage: Optional

- Cardinality:0..1

PV1 - Patient visit

- Usage: Required

- Cardinality:1..1

Seq.	Name	Opt.	Card.	Type	Table	Contents
1	Set ID - PV1	O	0..1	SI		
2	Patient Class	R	1..1	IS	HL70004	
3	Assigned Patient Location	O	0..1	PL		
3.1	point of care	O	0..	IS	HL70302	
3.2	room	O	0..	IS	HL70303	
3.3	bed	O	0..	IS	HL70304	
3.4	facility (HD)	O	0..	HD		



Seq.	Name	Opt.	Card.	Type	Table	Contents
3.4.1	namespace ID	O	0..	IS	HL70363	
3.4.2	universal ID	O	0..	ST		
3.4.3	universal ID type	O	0..	ID	HL70301	
4	Admission Type	O	0..1	IS	HL70007	
7	Attending Doctor	O	0..*	XCN	HL70010	
7.1	ID number (ST)	O	0..	ST		
7.2	family name	O	0..	FN		
7.2.1	surname	O	0..	ST		
7.2.2	own surname prefix	O	0..	ST		
7.2.3	own surname	O	0..	ST		
7.2.4	surname prefix from partner/spouse	O	0..	ST		
7.2.5	surname from partner/spouse	O	0..	ST		
7.3	given name	O	0..	ST		
7.4	second and further given names or initials thereof	O	0..	ST		
7.5	suffix (e.g., JR or III)	O	0..	ST		
7.6	prefix (e.g., DR)	O	0..	ST		
7.9	assigning authority	O	0..	HD		
7.9.1	namespace ID	O	0..	IS	HL70363	
7.9.2	universal ID	O	0..	ST		
7.9.3	universal ID type	O	0..	ID	HL70301	
8	Referring Doctor	O	0..*	XCN	HL70010	
8.1	ID number (ST)	O	0..	ST		
8.2	family name	O	0..	FN		
8.2.1	surname	O	0..	ST		
8.2.2	own surname prefix	O	0..	ST		
8.2.3	own surname	O	0..	ST		
8.2.4	surname prefix from partner/spouse	O	0..	ST		
8.2.5	surname from partner/spouse	O	0..	ST		
8.3	given name	O	0..	ST		
8.4	second and further given names or initials thereof	O	0..	ST		
8.5	suffix (e.g., JR or III)	O	0..	ST		
8.6	prefix (e.g., DR)	O	0..	ST		
8.9	assigning authority	O	0..	HD		
8.9.1	namespace ID	O	0..	IS	HL70363	
8.9.2	universal ID	O	0..	ST		
8.9.3	universal ID type	O	0..	ID	HL70301	



Seq.	Name	Opt.	Card.	Type	Table	Contents
10	Hospital Service	O	0..1	IS	HL70069	
11	Temporary Location	O	0..1	PL		
11.1	point of care	O	0..	IS	HL70302	
11.2	room	O	0..	IS	HL70303	
11.3	bed	O	0..	IS	HL70304	
11.4	facility (HD)	O	0..	HD		
11.4.1	namespace ID	O	0..	IS	HL70363	
11.4.2	universal ID	O	0..	ST		
11.4.3	universal ID type	O	0..	ID	HL70301	
14	Admit Source	O	0..1	IS	HL70023	
15	Ambulatory Status	O	0..*	IS	HL70009	
16	VIP Indicator	O	0..1	IS	HL70099	
17	Admitting Doctor	O	0..*	XCN	HL70010	
17.1	ID number (ST)	O	0..	ST		
17.2	family name	O	0..	FN		
17.2.1	surname	O	0..	ST		
17.2.2	own surname prefix	O	0..	ST		
17.2.3	own surname	O	0..	ST		
17.2.4	surname prefix from partner/spouse	O	0..	ST		
17.2.5	surname from partner/spouse	O	0..	ST		
17.3	given name	O	0..	ST		
17.4	second and further given names or initials thereof	O	0..	ST		
17.5	suffix (e.g., JR or III)	O	0..	ST		
17.6	prefix (e.g., DR)	O	0..	ST		
17.9	assigning authority	O	0..	HD		
17.9.1	namespace ID	O	0..	IS	HL70363	
17.9.2	universal ID	O	0..	ST		
17.9.3	universal ID type	O	0..	ID	HL70301	
18	Patient Type	O	0..1	IS	HL70018	
19	name validity range	R	1..1	CX		
19.1	ID	R	1..1	ST		
19.2	Check digit	O	0..	ST		
19.3	code identifying the check digit scheme employed	O	0..	ID	HL70061	
19.4	assigning authority	O	0..	HD		
19.4.1	namespace ID	O	0..	IS	HL70363	
19.4.2	universal ID	O	0..	ST		



Seq.	Name	Opt.	Card.	Type	Table	Contents
19.4.3	universal ID type	O	0..	ID	HL70301	
19.5	identifier type code (ID)	O	0..	ID	HL70203	
19.6	assigning facility	O	0..	HD		
19.6.1	namespace ID	O	0..	IS	HL70363	
19.6.2	universal ID	O	0..	ST		
19.6.3	universal ID type	O	0..	ID	HL70301	
19.7	effective date (DT)	O	0..	DT		
19.8	expiration date	O	0..	DT		
44	Admit Date/Time	O	0..1	TS		
44.1	Date/Time	O	0..	NM		e.g. 20040328134602.1234+0600
45	Discharge Date/Time	O	0..*	TS		
45.1	Date/Time	O	0..	NM		e.g. 20040328134602.1234+0600

1. Set ID - PV1

2. Patient Class

3. Assigned Patient Location

4. Admission Type

7. Attending Doctor

8. Referring Doctor

10. Hospital Service

11. Temporary Location

14. Admit Source

15. Ambulatory Status

16. VIP Indicator

17. Admitting Doctor

18. Patient Type

44.1. Date/Time

YYYYMMDD[HHMM[SS[.SSSS]]][+-ZZZZ]

45. Discharge Date/Time

45.1. Date/Time

YYYYMMDD[HHMM[SS[.SSSS]]][+-ZZZZ]



End of segment group VISIT

End of segment group PATIENT

Segment group: ORDER OBSERVATION

- Usage: Required
- Cardinality:1..*

ORC

- Usage: Required
- Cardinality:1..1

Seq.	Name	Opt.	Card.	Type	Table	Contents
1	Order Control	R	1..1	ID	HL70119	
2	Placer Order Number	R	1..1	EI		
2.1	entity identifier	R	1..1	ST		
2.2	namespace ID	O	0..	IS	HL70363	
3	Filler Order Number	R	1..1	EI		
3.1	entity identifier	R	1..1	ST		
3.2	namespace ID	O	0..	IS	HL70363	
5	Order Status	O	0..1	ID	HL70038	
7	Quantity/Timing	O	0..*	TQ		
7.4	start date/time	O	0..	TS		
7.4.1	Date/Time	R	1..1	NM		e.g. 20040328134602
7.5	end date/time	O	0..	TS		
7.5.1	Date/Time	R	1..1	NM		e.g. 20040328134602
7.6	priority	O	0..	ST		
10	Entered By	O	0..1	XCN		
10.1	ID number (ST)	O	0..	ST		
10.2	family name	O	0..	FN		
10.2.1	surname	O	0..	ST		
10.2.2	own surname prefix	O	0..	ST		
10.2.3	own surname	O	0..	ST		
10.2.4	surname prefix from partner/spouse	O	0..	ST		
10.2.5	surname from partner/spouse	O	0..	ST		
10.3	given name	O	0..	ST		
10.4	second and further given names or initials thereof	O	0..	ST		
10.5	suffix (e.g., JR or III)	O	0..	ST		
10.6	prefix (e.g., DR)	O	0..	ST		
10.7	degree (e.g., MD)	O	0..	IS	HL70360	



Seq.	Name	Opt.	Card.	Type	Table	Contents
10.8	source table	O	0..	IS	HL70297	
10.9	assigning authority	O	0..	HD		
10.9.1	namespace ID	O	0..	IS	HL70363	
10.9.2	universal ID	O	0..	ST		
10.9.3	universal ID type	O	0..	ID	HL70301	
10.10	name type code	O	0..	ID	HL70200	
10.11	identifier check digit	O	0..	ST		
10.12	code identifying the check digit scheme employed	O	0..	ID	HL70061	
10.13	identifier type code (IS)	O	0..	IS		
10.14	assigning facility	O	0..	HD		
10.14.1	namespace ID	O	0..	IS	HL70363	
10.14.2	universal ID	O	0..	ST		
10.14.3	universal ID type	O	0..	ID	HL70301	
10.15	Name Representation code	O	0..	ID	HL70465	
10.16	name context	O	0..	CE	HL70448	
10.16.1	identifier	O	0..	ST		
10.16.2	text	O	0..	ST		
10.16.3	name of coding system	O	0..	IS	HL70396	
10.16.4	alternate identifier	O	0..	ST		
10.16.5	alternate text	O	0..	ST		
10.16.6	name of alternate coding system	O	0..	IS	HL70396	
10.17	name validity range	O	0..	DR		
10.17.1	range start date/time	O	0..	TS		
10.17.2	range end date/time	O	0..	TS		
10.18	name assembly order	O	0..	ID	HL70444	
12	Ordering Provider	O	0..1	XCN		
12.1	ID number (ST)	C	0..	ST		
12.2	family name	O	0..	FN		
12.2.1	surname	O	0..	ST		
12.3	given name	O	0..	ST		
17	Entering Organization	O	0..1	CE		
17.1	identifier	R	1..1	ST		
17.2	text	O	0..	ST		
17.3	name of coding system	O	0..	IS	HL70396	
21	Ordering Facility Name	O	0..*	XON		
21.1	organization name	O	0..	ST		



Seq.	Name	Opt.	Card.	Type	Table	Contents
21.2	organization name type code	O	0..	IS	HL70204	
21.3	ID number (NM)	R	1..1	NM		
21.4	check digit	O	0..	NM		
21.5	code identifying the check digit scheme employed	O	0..	ID	HL70061	
21.6	assigning authority	O	0..	HD		
21.6.1	namespace ID	R	1..1	IS	HL70363	
21.7	identifier type code (IS)	O	0..	IS		
21.8	assigning facility ID	O	0..	HD	HL70300	
21.8.1	namespace ID	O	0..	IS	HL70363	
21.8.2	universal ID	O	0..	ST		
21.8.3	universal ID type	O	0..	ID	HL70301	
21.9	Name Representation code	O	0..	ID	HL70465	

1. Order Control

Fixed value: SC (Status changed)

2. Placer Order Number

3. Filler Order Number

3.2. namespace ID

If empty, the default assigning authority will be taken.

5. Order Status

Supported: IP (In Progress), CM (Completed), DC (Discontinued), CA (Cancelled), SC (Scheduled)

7. Quantity/Timing

Identical to the Quantity/Timing in OBR-Segment

7.4.1. Date/Time

If Agility is configured to send timezone information, datetime format

YYYYMMDD[HHHMM[SS]][+Z] is used otherwise YYYYMMDD[HHHMM [SS]] is used

7.5.1. Date/Time

If Agility is configured to send timezone information, datetime format

YYYYMMDD[HHHMM[SS]][+Z] is used otherwise YYYYMMDD[HHHMM [SS]] is used

7.6. priority

Supported: A (ASAP), R (Routine), PRN (As needed)



10. Entered By

12. Ordering Provider

12.1. ID number (ST)

the default identifier of the ordering provider

17. Entering Organization

17.2. text

If empty, the identifier will be taken as name.

17.3. name of coding system

If empty, the default requesting department will be taken

21. Ordering Facility Name

21.1. organization name

If empty, the identifier of the requesting facility will be taken as name.

21.6. assigning authority

If empty, the default assigning authority will be taken.

OBR

- Usage: Required

- Cardinality:1..1

Seq.	Name	Opt.	Card.	Type	Table	Contents
1	Set ID - OBR	R	1..1	SI		
2	Placer Order Number	R	1..1	EI		
2.1	entity identifier	R	1..1	ST		
2.2	namespace ID	O	0..	IS	HL70363	
2.3	universal ID	O	0..	ST		
2.4	universal ID type	O	0..	ID	HL70301	
3	Filler Order Number	R	1..1	EI		
3.1	entity identifier	R	1..1	ST		
3.2	namespace ID	O	0..	IS	HL70363	
3.3	universal ID	O	0..	ST		
3.4	universal ID type	O	0..	ID	HL70301	
4	Universal Service Identifier	R	1..1	CE		
4.1	identifier	R	1..1	ST		
4.2	text	O	0..	ST		
4.3	name of coding system	O	0..	IS	HL70396	
4.4	alternate identifier	R	1..1	ST		



Seq.	Name	Opt.	Card.	Type	Table	Contents
4.5	alternate text	O	0..	ST		
4.6	name of alternate coding system	O	0..	IS	HL70396	
5	Priority	O	0..1	ID		
6	Requested Date/Time	O	0..1	TS		
6.1	Date/Time	O	0..	NM		e.g. 20040328134602.1234+0600
7	Observation Date/Time #	O	0..1	TS		
7.1	Date/Time	R	1..1	NM		e.g. 20040328134602
13	Relevant Clinical Info.	O	0..1	ST		
16	Ordering Provider	O	0..1	XCN		
16.1	ID number (ST)	O	0..	ST		
16.2	family name	O	0..	FN		
16.2.1	surname	O	0..	ST		
16.3	given name	O	0..	ST		
18	Placer Field 1	R	1..1	ST		
19	Placer Field 2	R	1..1	ST		
20	Filler Field 1 +	R	1..1	ST		
24	Diagnostic Serv Sect ID	O	0..1	ID	HL70074	
25	Result Status +	R	1..1	ID	HL70123	
27	Quantity/Timing	O	0..*	TQ		
27.1	quantity	O	0..	CQ		
27.1.1	quantity	O	0..	NM		
27.1.2	units	O	0..	CE		
27.2	interval	O	0..	CM_RI		
27.2.1	repeat pattern	O	0..	IS	HL70335	
27.2.2	explicit time interval	O	0..	ST		
27.3	duration	O	0..	ST		
27.4	start date/time	O	0..	TS		
27.4.1	Date/Time	R	1..1	NM		e.g. 20040328134602
27.5	end date/time	O	0..	TS		
27.5.1	Date/Time	R	1..1	NM		e.g. 20040328134602
27.6	priority	O	0..	ST		
27.7	condition	O	0..	ST		
27.8	text (TX)	O	0..	TX		
27.9	conjunction component	O	0..	ID	HL70472	
27.10	order sequencing	O	0..	CM_OSD		
27.10.1	sequence/results flag	O	0..	ID		
27.10.2	placer order number: entity identifier	O	0..	ST		



Seq.	Name	Opt.	Card.	Type	Table	Contents
27.10.3	placer order number: namespace ID	O	0..	IS		
27.10.4	filler order number: entity identifier	O	0..	ST		
27.10.5	filler order number: namespace ID	O	0..	IS		
27.10.6	sequence condition value	O	0..	ID		
27.10.7	maximum number of repeats	O	0..	NM		
27.10.8	placer order number: universal ID	O	0..	ST		
27.10.9	placer order number; universal ID type	O	0..	ID		
27.10.10	filler order number: universal ID	O	0..	ST		
27.10.11	filler order number: universal ID type	O	0..	ID		
27.11	occurrence duration	O	0..	CE		
27.11.1	identifier	O	0..	ST		
27.11.2	text	O	0..	ST		
27.11.3	name of coding system	O	0..	IS	HL70396	
27.11.4	alternate identifier	O	0..	ST		
27.11.5	alternate text	O	0..	ST		
27.11.6	name of alternate coding system	O	0..	IS	HL70396	
27.12	total occurrences	O	0..	NM		
31	Reason for Study	O	0..2	CE		
31.1	identifier	O	0..	ST		
31.2	text	O	0..	ST		
31.3	name of coding system	O	0..	IS	HL70396	
31.4	alternate identifier	O	0..	ST		
31.5	alternate text	O	0..	ST		
31.6	name of alternate coding system	O	0..	IS	HL70396	
32	Principal Result Interpreter +	O	0..1	CM_NDL		
32.1	name	O	0..	CN		
32.1.1	ID number (ST)	O	0..	ST		
32.1.2	family name	O	0..	FN		
32.1.3	given name	O	0..	ST		
32.1.4	second and further given names or initials thereof	B	..	ST		
32.4	point of care (IS)	O	0..	IS		
32.5	room	O	0..	IS	HL70303	
32.6	bed	O	0..	IS	HL70304	
32.7	facility (HD)	O	0..	HD		
32.7.1	namespace ID	O	0..	IS	HL70363	
32.7.2	universal ID	O	0..	ST		



Seq.	Name	Opt.	Card.	Type	Table	Contents
32.7.3	universal ID type	O	0..	ID	HL70301	
32.8	location status	O	0..	IS	HL70306	
32.9	person location type	O	0..	IS	HL70305	
32.10	building	O	0..	IS	HL70307	
32.11	floor	O	0..	IS	HL70308	
33	Assistant Result Interpreter +	O	0..*	CM_NDL		
33.1	name	O	0..	CN		
33.1.1	ID number (ST)	O	0..	ST		
33.1.2	family name	O	0..	FN		
33.1.3	given name	O	0..	ST		
33.10	building	B	..	IS	HL70307	
35	Transcriptionist +	O	0..1	CM_NDL		
35.1	name	O	0..	CN		
35.1.1	ID number (ST)	O	0..	ST		
35.1.2	family name	O	0..	FN		
35.1.3	given name	O	0..	ST		
35.2	start date/time	O	0..	TS		
35.2.1	Date/Time	R	1..1	NM		e.g. 20040328134602.1234+0600
35.3	end date/time	O	0..	TS		
35.3.1	Date/Time	R	1..1	NM		e.g. 20040328134602.1234+0600
35.4	point of care (IS)	O	0..	IS		
35.5	room	O	0..	IS	HL70303	
35.6	bed	O	0..	IS	HL70304	
35.7	facility (HD)	O	0..	HD		
35.7.1	namespace ID	O	0..	IS	HL70363	
35.7.2	universal ID	O	0..	ST		
35.7.3	universal ID type	O	0..	ID	HL70301	
35.8	location status	O	0..	IS	HL70306	
35.9	person location type	O	0..	IS	HL70305	
35.10	building	O	0..	IS	HL70307	
35.11	floor	O	0..	IS	HL70308	
47	Filler Supplemental Service Information	O	0..1	CE	HL70411	

1. Set ID - OBR

2. Placer Order Number

Identical to ORC-2



3. Filler Order Number

Identical to ORC-3

4. Universal Service Identifier

5. Priority

Supported: A (ASAP), R (Routine), PRN (As needed)

6. Requested Date/Time

6.1. Date/Time

If Agility is configured to send timezone information, datetime format

YYYYMMDD[HHHMM[SS]][+/-ZZZZ] is used otherwise YYYYMMDD[HHHMM [SS]] is used

7. Observation Date/Time

7.1. Date/Time

If Agility is configured to send timezone information, datetime format

YYYYMMDD[HHHMM[SS]][+/-ZZZZ] is used otherwise YYYYMMDD[HHHMM [SS]] is used

13. Relevant Clinical Info.

16. Ordering Provider

same as ORC-12

16.1. ID number (ST)

the default identifier of the ordering provider

18. Placer Field 1

19. Placer Field 2

20. Filler Field 1 +

24. Diagnostic Serv Sect ID

25. Result Status +

Supported: F (final), P (Preliminary), C (Correction), D (Deleted)

27. Quantity/Timing

Identical to ORC-7

27.4.1. Date/Time

If Agility is configured to send timezone information, datetime format

YYYYMMDD[HHHMM[SS]][+/-ZZZZ] is used otherwise YYYYMMDD[HHHMM [SS]] is used

27.5.1. Date/Time

If Agility is configured to send timezone information, datetime format

YYYYMMDD[HHHMM[SS]][+/-ZZZZ] is used otherwise YYYYMMDD[HHHMM [SS]] is used



27.6. priority

Supported: A (ASAP), R (Routine), PRN (As needed)

31. Reason for Study

32. Principal Result Interpreter +

The segment contains:

- the validator when the report is final
- the author when the report is preliminary

32.1.1. ID number (ST)

The default identifier of the principal result interpreter

33. Assistant Result Interpreter +

This segment contains:

- every contributor when preliminary report
- the author (when different from validator) and every contributor when final report

33.1.1. ID number (ST)

the default identifier of the assistant result interpreter

35. Transcriptionist +

35.1.1. ID number (ST)

the default identifier of the transcriptionist

35.2.1. Date/Time

YYYYMMDD[HHMM[SS[.SSSS]]][+ZZZZ]

35.3.1. Date/Time

YYYYMMDD[HHMM[SS[.SSSS]]][+ZZZZ]

47. Filler Supplemental Service Information

ZDS

- Usage: Optional
- Cardinality:0..1

Seq.	Name	Opt.	Card.	Type	Table	Contents
1	Reference Pointer	R	1..1	ST		
2	Application ID	O	0..1	ST		e.g. AGILITY
3	Type of Data	O	0..1	ST		e.g. Application
4	Subtype	O	0..1	ST		e.g. DICOM



1. Reference Pointer

the Agility Study instance UID

2. Application ID

fixed value == AGILITY

3. Type of Data

fixed value == Application

4. Subtype

fixed value == DICOM

DG1 - Diagnosis

- Usage: Optional

- Cardinality:0..*

Seq.	Name	Opt.	Card.	Type	Table	Contents
1	Set ID - DG1	R	1..1	SI		
2	Diagnosis Coding Method	O	0..1	ID	HL70053	
3	Diagnosis Code - DG1	R	1..1	CE	HL70051	
3.1	identifier	R	1..1	ST		
3.2	text	R	1..1	ST		
3.3	name of coding system	R	1..1	IS	HL70396	
3.4	alternate identifier	R	1..1	ST		
3.5	alternate text	O	0..	ST		
3.6	name of alternate coding system	O	0..	IS	HL70396	
4	Diagnosis Description	B	0..*	ST		
5	Diagnosis Date/Time	R	1..1	TS		
5.1	Date/Time	R	1..1	NM		e.g. 20040328134602
6	Diagnosis Type	R	1..1	IS	HL70052	
7	Major Diagnostic Category	B	0..*	CE	HL70118	
7.1	identifier	O	0..	ST		
7.2	text	O	0..	ST		
7.3	name of coding system	O	0..	IS	HL70396	
7.4	alternate identifier	O	0..	ST		
7.5	alternate text	O	0..	ST		
7.6	name of alternate coding system	O	0..	IS	HL70396	
8	Diagnostic Related Group	B	0..*	CE	HL70055	
8.1	identifier	O	0..	ST		
8.2	text	O	0..	ST		
8.3	name of coding system	O	0..	IS	HL70396	



Seq.	Name	Opt.	Card.	Type	Table	Contents
8.4	alternate identifier	O	0..	ST		
8.5	alternate text	O	0..	ST		
8.6	name of alternate coding system	O	0..	IS	HL70396	
9	DRG Approval Indicator	B	0..*	ID	HL70136	
10	DRG Grouper Review Code	B	0..*	IS	HL70056	
11	Outlier Type	B	0..*	CE	HL70083	
11.1	identifier	O	0..	ST		
11.2	text	O	0..	ST		
11.3	name of coding system	O	0..	IS	HL70396	
11.4	alternate identifier	O	0..	ST		
11.5	alternate text	O	0..	ST		
11.6	name of alternate coding system	O	0..	IS	HL70396	
12	Outlier Days	B	0..*	NM		
13	Outlier Cost	B	0..*	CP		
13.1	price	O	0..	MO		
13.1.1	quantity	O	0..	NM		
13.1.2	denomination	O	0..	ID		
13.2	price type	O	0..	ID	HL70205	
13.3	from value	O	0..	NM		
13.4	to value	O	0..	NM		
13.5	range units	O	0..	CE		
13.5.1	identifier	O	0..	ST		
13.5.2	text	O	0..	ST		
13.5.3	name of coding system	O	0..	IS	HL70396	
13.5.4	alternate identifier	O	0..	ST		
13.5.5	alternate text	O	0..	ST		
13.5.6	name of alternate coding system	O	0..	IS	HL70396	
13.6	range type	O	0..	ID	HL70298	
14	Grouper Version And Type	B	0..*	ST		
15	Diagnosis Priority	O	0..1	ID	HL70359	
16	Diagnosing Clinician	O	0..*	XCN		
16.1	ID number (ST)	O	0..	ST		
16.2	family name	O	0..	FN		
16.2.1	surname	O	0..	ST		
16.2.2	own surname prefix	O	0..	ST		
16.2.3	own surname	O	0..	ST		
16.2.4	surname prefix from partner/spouse	O	0..	ST		



Seq.	Name	Opt.	Card.	Type	Table	Contents
16.2.5	surname from partner/spouse	O	0..	ST		
16.3	given name	O	0..	ST		
16.4	second and further given names or initials thereof	O	0..	ST		
16.5	suffix (e.g., JR or III)	O	0..	ST		
16.6	prefix (e.g., DR)	O	0..	ST		
16.7	degree (e.g., MD)	O	0..	IS	HL70360	
16.8	source table	O	0..	IS	HL70297	
16.9	assigning authority	O	0..	HD		
16.9.1	namespace ID	O	0..	IS	HL70363	
16.9.2	universal ID	O	0..	ST		
16.9.3	universal ID type	O	0..	ID	HL70301	
16.10	name type code	O	0..	ID	HL70200	
16.11	identifier check digit	O	0..	ST		
16.12	code identifying the check digit scheme employed	O	0..	ID	HL70061	
16.13	identifier type code (IS)	O	0..	IS		
16.14	assigning facility	O	0..	HD		
16.14.1	namespace ID	O	0..	IS	HL70363	
16.14.2	universal ID	O	0..	ST		
16.14.3	universal ID type	O	0..	ID	HL70301	
16.15	Name Representation code	O	0..	ID	HL70465	
16.16	name context	O	0..	CE	HL70448	
16.16.1	identifier	O	0..	ST		
16.16.2	text	O	0..	ST		
16.16.3	name of coding system	O	0..	IS	HL70396	
16.16.4	alternate identifier	O	0..	ST		
16.16.5	alternate text	O	0..	ST		
16.16.6	name of alternate coding system	O	0..	IS	HL70396	
16.17	name validity range	O	0..	DR		
16.17.1	range start date/time	O	0..	TS		
16.17.2	range end date/time	O	0..	TS		
16.18	name assembly order	O	0..	ID	HL70444	
17	Diagnosis Classification	O	0..1	IS	HL70228	
18	Confidential Indicator	O	0..1	ID	HL70136	
19	Attestation Date/Time	O	0..1	TS		
19.1	Date/Time	R	1..1	NM		e.g. 20040328134602.1234+0600



1. Set ID - DG1

2. Diagnosis Coding Method

3. Diagnosis Code - DG1

4. Diagnosis Description

5. Diagnosis Date/Time

5.1. Date/Time

If Agility is configured to send timezone information, datetime format
YYYYMMDD[HHHMM[SS]][+-ZZZZ] is used otherwise YYYYMMDD[HHHMM [SS]] is used

6. Diagnosis Type

7. Major Diagnostic Category

8. Diagnostic Related Group

9. DRG Approval Indicator

10. DRG Grouper Review Code

11. Outlier Type

12. Outlier Days

13. Outlier Cost

14. Grouper Version And Type

15. Diagnosis Priority

16. Diagnosing Clinician

17. Diagnosis Classification

18. Confidential Indicator

19. Attestation Date/Time

19.1. Date/Time

YYYYMMDD[HHHMM[SS[.SSSS]]][+- ZZZZ]



Segment group: OBSERVATION - PATIENT RESULT.ORDER OBSERVATION.OBSERVATION

- Usage: Optional
- Cardinality:0..*
- Implementation note: Segment that repeats for every report and addenda ONLY in the last ORDER_OBSERVATION will there be an OBSERVATION segment

OBX - Observation/Result

- Usage: Required
- Cardinality:1..1

Seq.	Name	Opt.	Card.	Type	Table	Contents
1	Set ID - OBX	O	0..1	SI		
2	Value Type	O	0..1	ID	HL70125	e.g. FT
3	Observation Identifier	R	1..1	CE		
3.1	identifier	R	1..1	ST		
3.2	text	O	0..	ST		
3.3	name of coding system	O	0..	IS	HL70396	
3.4	alternate identifier	O	0..	ST		
4	Observation Sub-Id	R	1..1	ST		
5	Observation Value	O	0..1	*		
11	Observation Result Status	R	1..1	ID	HL70085	e.g. F
14	Date/Time of the Observation	O	0..1	TS		
14.1	Date/Time	R	1..1	NM		e.g. 20080319103922
16	Responsible Observer	O	0..1	XCN		
16.1	ID number (ST)	O	0..	ST		
16.2	family name	O	0..	FN		
16.2.1	surname	O	0..	ST		
16.3	given name	O	0..	ST		

1. Set ID - OBX

The sequence number

2. Value Type

Formatted text (FT) for report content
Reference Pointer (RP) for attachments

3. Observation Identifier

3.1. identifier

For FT: The procedure definition code followed by either '&ADT' for addenda of the report and '&GDT' for every other type or report
For RP: attachment code



3.2. text

For FT: The name of the procedure definition

For RP: attachment description

3.3. name of coding system

fixed value == AGILITY

3.4. alternate identifier

For RP: accession number where attachment belongs to. In case of order/patient level attachment, this will be empty

4. Observation Sub-Id

for FT: the name of the section of the report or addenda

for RP: unique key identifying a specific attachment (primary key of attachment)

5. Observation Value

For FT: the content of the section line

For RP, datatype is ST: the reference pointer to the attachment (taking into account the value from the 'attachment path' parameter on the pipeline fa_ade)

11. Observation Result Status

For FT Supported: F (final), P (Preliminary), C (Correction), D (Deleted)

For RP: Fixed value O

14. Date/Time of the Observation

The date of the observation

14.1. Date/Time

If Agility is configured to send timezone information, datetime format

YYYYMMDD[HHMM[SS]][+ZZZZ] is used otherwise YYYYMMDD[HHMM [SS]] is used

16. Responsible Observer

This segment contains:

In case the OBX-segment contains report content:

- the author when the report is preliminary
- the validator when the report is final

In case the OBX-segment contains addendum content:

- the author of this particular addendum content when this addendum is preliminary
- the validator of this particular addendum content when this addendum is final

16.1. ID number (ST)

The default identifier of the responsible observer

End of segment group OBSERVATION

End of segment group ORDER_OBSERVATION



End of segment group PATIENT_RESULT